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>> EMC Guidance and Manufacturer's declaration

8.2 EMC

1. Guidance and Manufacturer's Declaration.

| | | |
|--|-------------------------|--|
| Electromagnetic emission ⁴ | | Electromagnetic environment - guidance ⁴ |
| The Mobile ENT Unit(Model : NET-3000, NET-600, NET-1100) is intended for use in electromagnetic environment specified below. The customer or the user of the Mobile ENT Unit(Model : NET-3000, NET-600, NET-1100) should assure that it is used in such an environment. ⁴ | | |
| Emission test ² | Compliance ² | Electromagnetic environment - guidance ⁴ |
| RF emissions – CISPR11 ² | Group 1 ² | The Mobile ENT Unit(Model : NET-3000, NET-600, NET-1100) uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. ⁴ |
| RF emissions- CISPR11 ² | Class A ² | The Mobile ENT Unit(Model : NET-3000, NET-600, NET-1100) is suitable for use in all establishments (i.e. hospitals, doctors practice etc.) other than domestic. Mobile ENT Unit(Model : NET-3000, NET-600, NET-1100) is intended for professional use only. ⁴ |
| Harmonic emissions IEC 61000-3-2 ² | Class A ² | |
| Voltage fluctuations/flicker emissions IEC6100-3-3 ² | Complies ² | |

| | | | |
|--|--|--|---|
| Electromagnetic immunity ⁴ | | | |
| The Mobile ENT Unit(Model : NET-3000, NET-600, NET-1100) is intended for use in the electromagnetic environment specified below. The customer or the user of the Mobile ENT Unit(Model : NET-3000, NET-600, NET-1100) should assure that it is used in such an environment. ⁴ | | | |
| IMMUNITY test ¹ | IEC 60601 ⁴ test level ³ | Compliance level ¹ | Electromagnetic environment Guidance ⁴ |
| Electrostatic discharge (ESD) ⁴ IEC 61000-4-2 ² | + 5 kV contact ⁴ ± 3 kV air ² | + 6 kV contact ⁴ ± 8 kV air ² | Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %. ² |
| Electrical fast transient/burst ⁴ IEC 61000-4-4 ² | ± 2 kV for power supply lines ⁴ ± 1 kV for input/output lines ² | ± 2 kV for A.C. power ports ⁴ ± 2 kV for D.C. power ports ⁴ ± 1 kV for Signal ports ² | Mains power quality should be that of a typical commercial or hospital environment. ² |
| Surge ⁴ IEC 61000-4-5 ² | + 1 kV line(s) to line(s) ⁴ ± 2 kV line(s) to earth ² | 0.5 kV and 1.0 kV (Line to line) ⁴ 0.5 kV, 1.0 kV and 2.0 kV (Line to earth) ² | Mains power quality should be that of a typical commercial or hospital environment. ² |

>> EMC Guidance and Manufacturer's declaration

| | | | |
|--|---|---|--|
| Voltage dips, short-interruptions and voltage variations on power supply input lines IEC 61000-4-11 | <5 % UT (>95 % dip in UT) for 0.5 cycle 40 % UI (60 % dip in UT) for 5 cycles 70 % UT (30 % dip in UT) for 25 cycles <5 % UT (>95 % dip in UT) for 5 s | <5 % UT (>95 % dip in UT) for 0.5 cycle 40 % UI (60 % dip in UT) for 5 cycles 70 % UT (30 % dip in UT) for 25 cycles <5 % UT (>95 % dip in UT) for 5 s | Mains power quality should be that of a typical commercial or hospital environment. If the user of the MF EQUIPMENT or MF SYSTEM requires continued operation during power mains interruptions, it is recommended that the [MF EQUIPMENT or ME SYSTEM] be powered from an uninterruptible power supply or a battery. |
| Power frequency magnetic field (50/60 Hz) IEC 61000-4-8 | 3 A/m | 3 A/m | Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment. |
| NOTE UI is the ac. mains voltage prior to application of the test level. | | | |

| Electromagnetic immunity | | | |
|---|----------------------|------------------|---|
| The <i>Mobile ENT Unit(Model : NET-3000, NET-600, NET-1100)</i> is intended for use in the electromagnetic environment specified below. The customer or the user of the <i>Mobile ENT Unit(Model : NET-3000, NET-600, NET-1100)</i> should assure that it is used in such an environment. | | | |
| IMMUNITY test | IEC 60601 TEST LEVEL | Compliance level | Electromagnetic environment – guidance |
| | | | Portable and mobile RF communications equipment should be used no closer to any part of the <i>Mobile ENT Unit(Model : NET-3000, NET 600, NET 1100)</i> , including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. |
| | | | Recommended separation distance |
| Conducted RF | 3 Vrms | 3 Vrms [VI] | $d = 1.2\sqrt{P}$ (Conducted) |
| IEC 61000-4-6 | 150 kHz to 80 MHz | | $d = 1.2\sqrt{P}$ 80MHz to 800 MHz (Radiated) |
| Radiated RF | 3 V/m | 3 V/m [E1] | $d = 2.3\sqrt{P}$ 800MHz to 2,5 GHz (Radiated) |
| IEC 61000-4-3 | 80 MHz to 2,5GHz | | where P is the maximum output power |

>> EMC Guidance and Manufacturer's declaration

| Recommended separation distances between ¹⁾ | | | |
|--|--|---|--|
| portable and mobile RF communications equipment and the <i>Mobile ENT Unit(Model : NET-3000, NET-600, NET-1100)</i> . | | | |
| The <i>Mobile ENT Unit(Model : NET 3000, NET 600, NET 1100)</i> is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the <i>RS-6000</i> can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the <i>Mobile ENT Unit(Model : NET-3000, NET-600, NET-1100)</i> as recommended below, according to the maximum output power of the communications equipment. | | | |
| Rated maximum output power of transmitter., W., | Separation distance according to frequency of transmitter., m., | | |
| | 150 kHz to 80 MHz., $d = 1.2\sqrt{P}$., | 80 MHz to 800 MHz., $d = 1.2\sqrt{P}$., | 800 MHz to 2,5 GHz., $d = 2.3\sqrt{P}$., |
| 0.01., | 0.12., | 0.12., | 0.23., |
| 0.1., | 0.38., | 0.38., | 0.73., |
| 1., | 1.2., | 1.2., | 2.3., |
| 10., | 3.8., | 3.8., | 7.3., |
| 100., | 12., | 12., | 23., |
| For transmitters rated at a maximum output power not listed above, the recommended separation distance <i>d</i> in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where <i>P</i> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer. | | | |
| NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. | | | |
| NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected | | | |

| Recommended separation distances between portable and mobile RF communications equipment and the <i>Mobile ENT Unit(Model : NET-3000, NET-600, NET-1100)</i> . | | | |
|--|--|---|--|
| The <i>Mobile ENT Unit(Model : NET-3000, NET-600, NET-1100)</i> is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the <i>RS-6000</i> can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the <i>Mobile ENT Unit(Model : NET-3000, NET-600, NET-1100)</i> as recommended below, according to the maximum output power of the communications equipment. | | | |
| Rated maximum output power of transmitter., W., | Separation distance according to frequency of transmitter., m., | | |
| | 150 kHz to 80 MHz., $d = 1.2\sqrt{P}$., | 80 MHz to 800 MHz., $d = 1.2\sqrt{P}$., | 800 MHz to 2,5 GHz., $d = 2.3\sqrt{P}$., |
| 0.01., | 0.12., | 0.12., | 0.23., |
| 0.1., | 0.38., | 0.38., | 0.73., |
| 1., | 1.2., | 1.2., | 2.3., |
| 10., | 3.8., | 3.8., | 7.3., |
| 100., | 12., | 12., | 23., |
| For transmitters rated at a maximum output power not listed above, the recommended separation distance <i>d</i> in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where <i>P</i> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer. | | | |
| NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. | | | |
| NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people. | | | |